



EZ-IO







Intraosseous Infusion System

To Insert Needle Set:

1. Protect yourself (BSI)
2. Identify indication
3. Check for contraindication
4. Locate landmarks **1***
5. Clean site **2**
6. Prepare driver and needle set
7. Stabilize leg
8. Insert EZ-IO® needle set **3**
9. Remove driver from needle set
10. Remove stylet from catheter **4**
11. Confirm placement
12. Attach EZ-Connect™
13. Inject IO 20-40 mg of 2% Lidocaine in alert patients
14. Syringe bolus (flush) IO with 10 ml NS **5**
15. Start infusion under pressure **6**
16. Secure tubing and catheter

*additional sites available

Do Not Leave the EZ-IO catheter in for more than 24 hours.

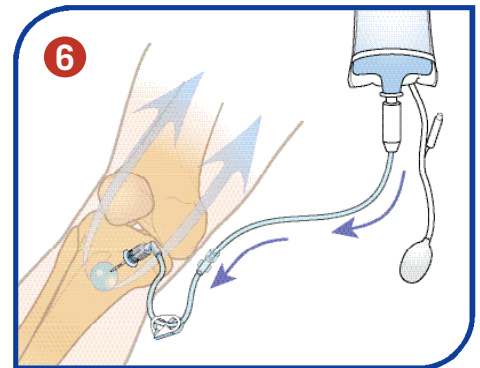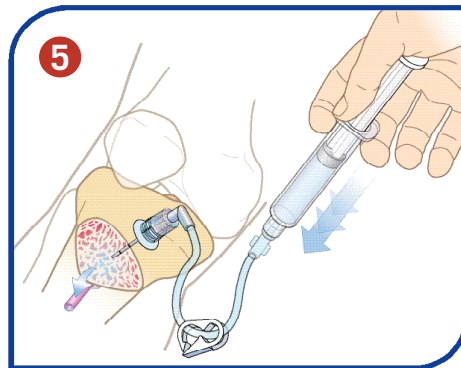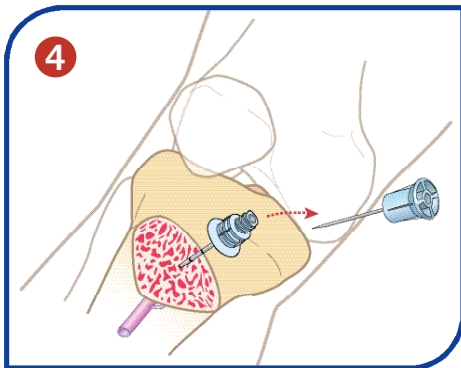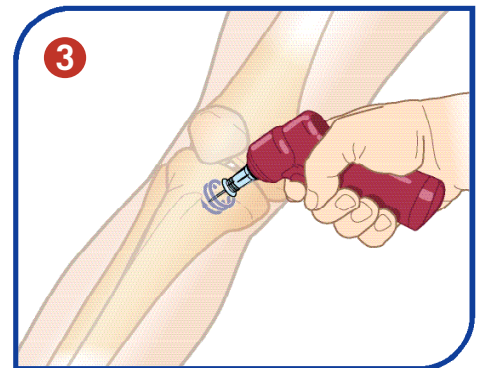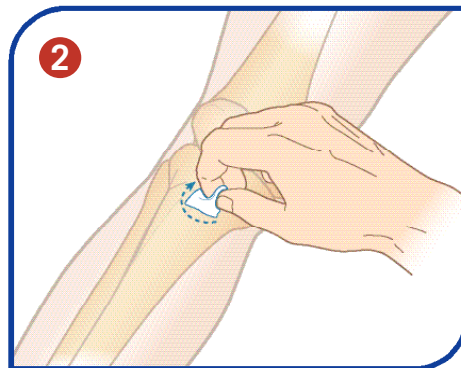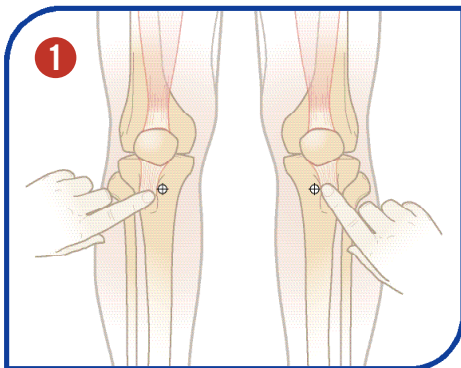

vidacare

722 Isom Road, San Antonio, TX 78216 • (866) 479 8500 • www.vidacare.com

T-420 Rev A

Immediate Vascular Access...
*When You Need It Most*SM