



## EZ-IO


### Intraosseous Infusion System

#### To Remove Catheter:

- Stabilize patient's extremity
- Connect sterile Luer lock syringe to hub of catheter (if available)
- Rotate catheter clockwise – while gently pulling
- When catheter has been removed, immediately place in appropriate biohazard container

**DO NOT ROCK** the catheter while removing. Rocking or bending the catheter with a syringe may cause the catheter to separate from the hub!

**Do not leave the EZ-IO catheter in place for more than 24 hours**

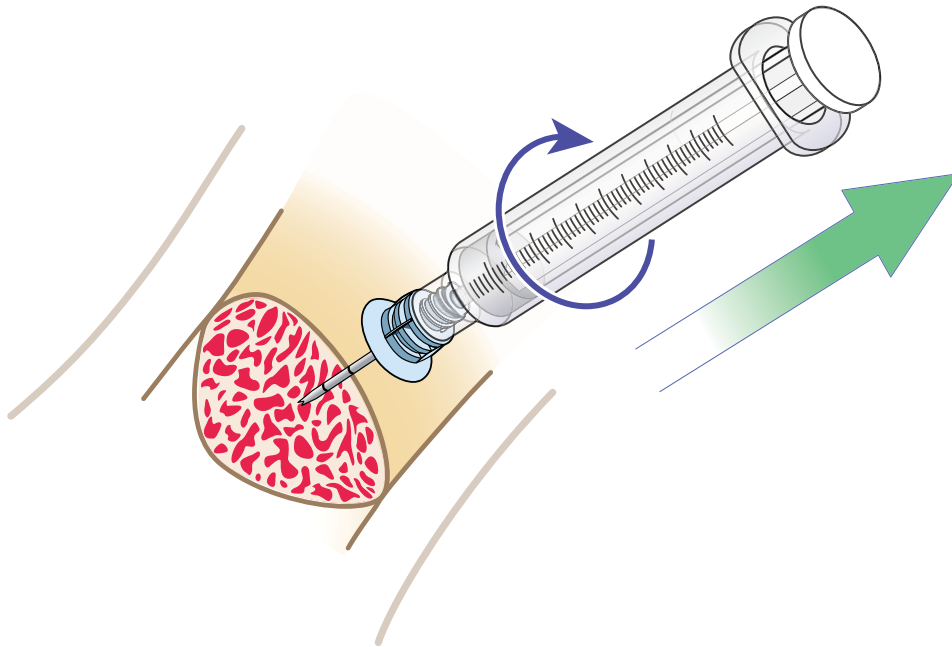

**Emergency contact in US or Canada call: 1 (800) 680 4911**

**[www.vidacare.com](http://www.vidacare.com)**

**vidacare®**

722 Isom Road, San Antonio, TX 78216  
(866) 479 8500

T-400 Rev E

Immediate Vascular Access...  
*When You Need It Most<sup>SM</sup>*